



Product Specifications

LifeVac Travel Kit

- **Life-Saving Anti-Choking Device** – At time of publication, the LifeVac has contributed to saving over 5,000 lives and counting, offering critical support during choking emergencies on the go.
- **Four Masks Included** – Features paediatric, child/small adult, adult, plus a practice mask to ensure confidence and readiness while travelling.
- **Regulated Medical Device** – CE marked, UKCA marked, MHRA regulated and MDR compliant
- **One-way valve and suction system** – Offers non-invasive airway clearance while reducing the risk of pushing obstructions deeper, protecting both the casualty and responders
- **Compact and Portable Design** – Lightweight and space-efficient, the travel LifeVac kit easily fits into luggage, backpacks, travel bags or even a car boot or glove compartment for quick access during meals or outings.

The **LifeVac Travel Kit** has demonstrated its effectiveness through numerous case studies and testimonials. It offers valuable peace of mind when dining out or travelling, ensuring you're prepared should an emergency occur. The compact carry case that is supplied with the travel kit ensures that both the device and masks are kept together and remain clean no matter where they are stored on your travels. The kit fits comfortably in a backpack, glove compartment, or even in your carry-on luggage when flying. **Key Benefits of The LifeVac Anti-Choking Device**

- **Includes multiple mask sizes** – the LifeVac Travel Kit offers three different masks to ensure a secure seal for both paediatric and adult users on the go.
- **Practice mask provided** – a dedicated mask is included to help users build confidence and practise device use before travelling.
- **Designed for portability** – compact and lightweight, it fits easily into luggage, backpacks, or travel bags for convenient access anywhere.
- **Regulated Medical Device** – CE marked, UKCA marked, MHRA regulated and MDR compliant for peace of mind while travelling.
- **One-way valve and suction system** – offers non-invasive airway clearance while reducing the risk of pushing obstructions deeper, protecting both the casualty and responders.
- **No batteries or power required** – always ready for immediate use, even in remote areas or while travelling.
- **Eligible for free replacement** – if used, a replacement device can be requested by submitting a form on the LifeVac website.

The Kit Includes:

- LifeVac Instruction Booklet – The instruction booklet details how to use LifeVac, and which coloured sticker identifies the size of the mask.
- LifeVac Unit.
- 1 Paediatric Mask.
- 1 Small Child/Adult Mask.
- 1 Medium Adult Test Mask.
- 1 Large Adult Mask.
- 1 Compact Yellow Travel Bag
- 1 Window/Door Sticker – to easily identify where a LifeVac is stored.
- Feedback Form – The Feedback Form is to be filled out once your LifeVac is used to help us raise awareness and improve our service.

How to Use the LifeVac

- Push and twist the mask into the base of your LifeVac unit.
- Hold the mask firmly in place over their mouth and nose with your less dominant hand, making sure their head is tilted back.
- Use your dominant hand on the LifeVac handle, depress and then pull upwards with a short, swift tug (while holding the mask firmly in place).
- Roll the person onto their side, check their mouth to clear any debris, and repeat the process up to five times if the obstruction is not cleared.



How Does It Work? LifeVac features two patented discs that accommodate interchangeable mask sizes, recognising that one size does not fit all. It also incorporates a patented one-way valve system that prevents air from being forced through the mask during use. LifeVac's non-invasive design ensures that no tubes can push the tongue back or drive the obstruction deeper during a panic situation. This design also prevents tubes from becoming blocked, which could otherwise render the device ineffective if only part of the obstruction has been cleared. **Ideal For Travel**

Locations

- Airplane cabins and airport lounges
- Hotels and resorts
- Cruise ships and ferries
- Vacation rentals and holiday homes
- Camping sites and RVs
- Train and bus stations
- Conference centres and event venues
- Restaurants and cafes in travel hubs

Product Specification Weight: 8 oz Dimensions: 35.6 x 20.3 x 7.6cm



+ LifeVac® Choking Rescue Contents

Items	Qty
Paediatric Mask	1
Small Child/Adult Mask	1
Practice Mask (Medium Adult)	1
Large Adult Mask	1
LifeVac® Anti-Choking Device	1
Window/Door Sticker	1
LifeVac® Instruction Booklet	1
Travel Bag	1


