





Product Specifications

HypaSoothe Burns Kit with Vinyl Wallet - Medium

- Portable burns kit in vinyl pouch
- Ideal for providing immediate first aid to minor burn injuries
- Supplied in a compact vinyl wallet
- Contains HypaSoothe burn dressings and gel which require no pre-cooling
- Kit Dimensions: 22cm x 27cm

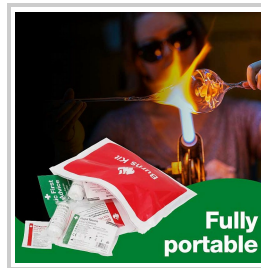


HypaSoothe Burns Kits in Vinyl Wallets include a range of products designed to provide effective treatment of burn injuries and include HypaSoothe burn dressings and gel which require no pre-cooling. The kits come in vinyl wallets and are lightweight, highly portable and available in 3 sizes (small, medium and large).

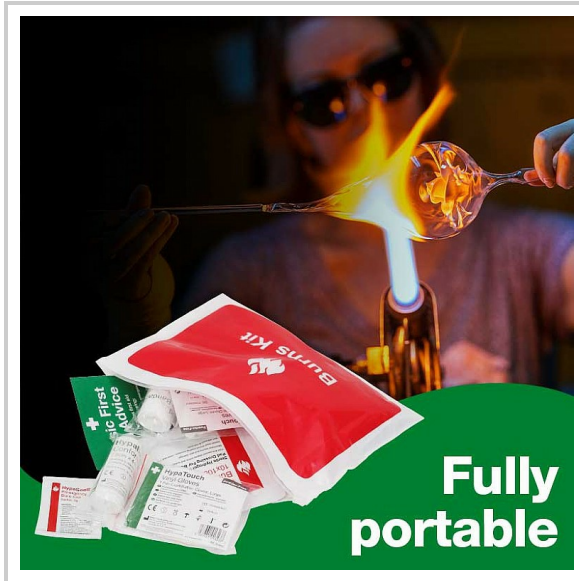



HypaSoothe Burns Contents

Items	Qty
First Aid Guidance Leaflet	1
HypaSoothe Burn Dressings, 10x10cm	2
HypaSoothe Burn Dressing, 20x20cm	1
HypaSoothe Emergency Burn Gel Sachets, 6g	6
HypaBand Safety Pins	12
HypaBand Conforming Bandages	3
Snips Clothing Cutters, 15cm	1
HypaTouch Disposable Gloves (Pair)	3

Treating burns injuries



Cooling with Water
If running water is available, immerse the burn in it for 10 minutes or until the burning sensation has stopped.

Remove
Gently remove any rings, watches, belts, shoes or any clothing from the injured area. If clothing is burnt onto the wound, DO NOT pull off!

Dressing a Wound
If the burn has been cooled with water, a sterile non-sticky dressing should be applied.

Applying a Burn Dressing
Burn dressings can also be used to treat burn injuries. Some burn dressings require the burn to be cooled prior to applying the dressing whilst others can be applied without pre-cooling. Check usage instructions on the dressing before use.

Casualties should be referred to hospital in the following instances:

- Superficial burn covering more than 5% of the body's surface
- Partial thickness burn covering more than 1% of the body
- Full thickness burns
- Burns involving children/ elderly
- Burns involving feet, hands, face or genital areas
- Burns that extend around a limb
- Burns with a mixed pattern of depth
- If unsure of depth or severity of burn

Effective treatment for burn injuries

Treating burns injuries



Cooling with Water
If running water is available, immerse the burn in it for 10 minutes or until the burning sensation has stopped.

Remove
Gently remove any rings, watches, belts, shoes or any clothing from the injured area. If clothing is burnt onto the wound, DO NOT pull off!

Dressing a Wound
If the burn has been cooled with water, a sterile non-sticky dressing should be applied.

Applying a Burn Dressing
Burn dressings can also be used to treat burn injuries. Some burn dressings require the burn to be cooled prior to applying the dressing whilst others can be applied without pre-cooling. Check usage instructions on the dressing before use.

Casualties should be referred to hospital in the following instances:

- Superficial burn covering more than 5% of the body's surface
- Partial thickness burn covering more than 1% of the body
- Full thickness burns
- Burns involving children/ elderly
- Burns involving feet, hands, face or genital areas
- Burns that extend around a limb
- Burns with a mixed pattern of depth
- If unsure of depth or severity of burn

HypaSoothe Burn Dressing 10x10cm
Sterile Hydrogel Impregnated Pad Dressing For Burns