

ISSUE: How to disinfect masks and filters to prevent the Coronavirus.

Productos Climax recommends the following instructions to clean and disinfectant our equipment

Full face masks and half mask

Remove the inner mask, valves and the filters

Clean face piece by immersing in water temperature not to exceed 40°C, and scrub with soft brush until clean. Add neutral detergent if necessary. Do not use cleaners containing lanolin or other oils.

Dry the mask with a soft, clean cloth or make it dry naturally.

Disinfect using a hydroalcoholic solution. Wait 5 min before put it on.

Filters

Disinfect the filter casing using a hydroalcoholic solution. Wait 5 min before attach it on the mask.

Parets del Vallès, 03/04/20



Javier Martín

Responsable Calidad/ Quality Management Manager