


LifeVac Wall-Mounted Kit in Evolution Case

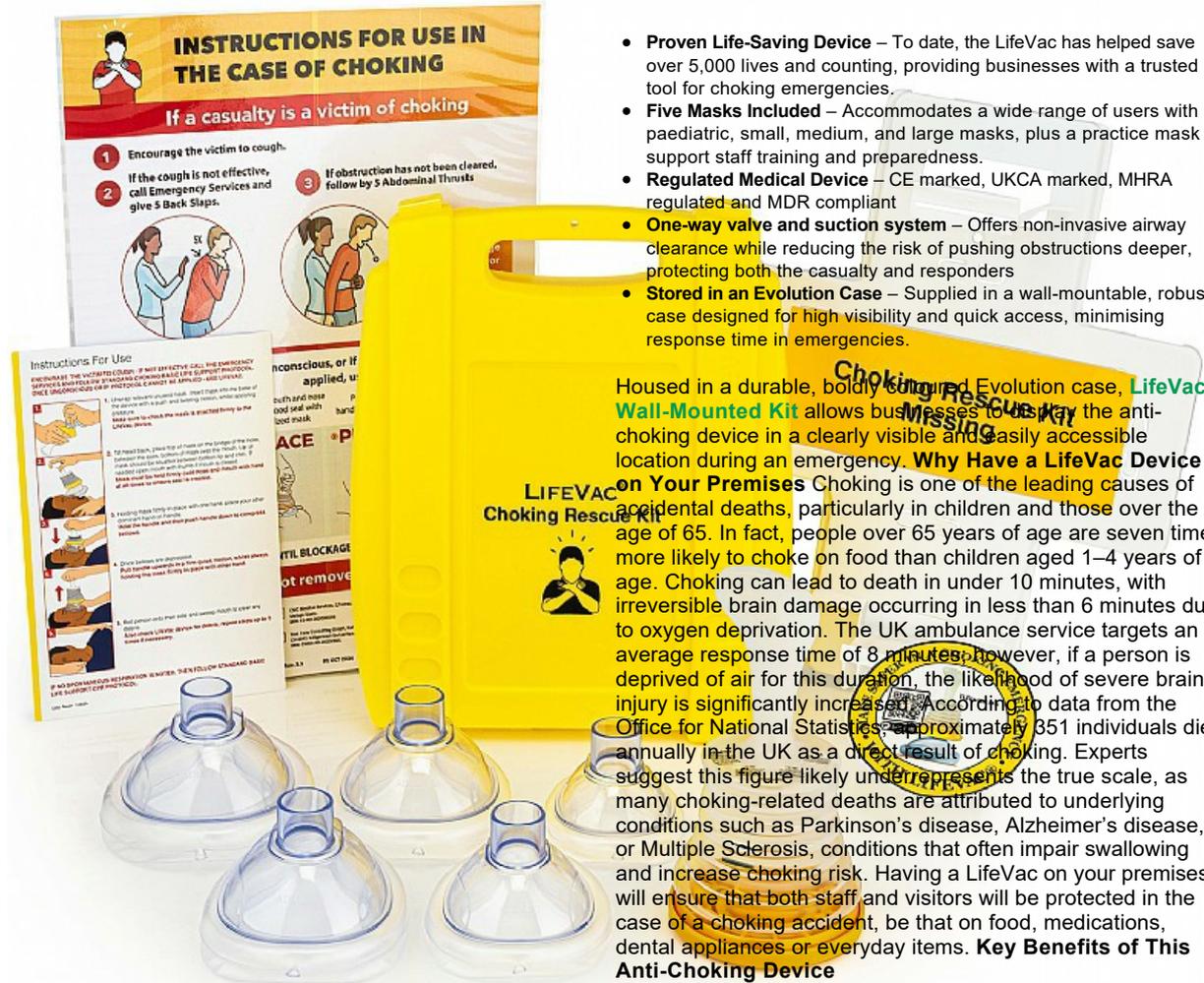

- **Proven Life-Saving Device** – To date, the LifeVac has helped save over 5,000 lives and counting, providing businesses with a trusted tool for choking emergencies.
- **Five Masks Included** – Accommodates a wide range of users with paediatric, small, medium, and large masks, plus a practice mask to support staff training and preparedness.
- **Regulated Medical Device** – CE marked, UKCA marked, MHRA regulated and MDR compliant
- **One-way valve and suction system** – Offers non-invasive airway clearance while reducing the risk of pushing obstructions deeper, protecting both the casualty and responders
- **Stored in an Evolution Case** – Supplied in a wall-mountable, robust case designed for high visibility and quick access, minimising response time in emergencies.

Housed in a durable, boldly coloured Evolution case, **LifeVac Wall-Mounted Kit** allows businesses to display the anti-choking device in a clearly visible and easily accessible location during an emergency. **Why Have a LifeVac Device on Your Premises** Choking is one of the leading causes of accidental deaths, particularly in children and those over the age of 65. In fact, people over 65 years of age are seven times more likely to choke on food than children aged 1–4 years of age. Choking can lead to death in under 10 minutes, with irreversible brain damage occurring in less than 6 minutes due to oxygen deprivation. The UK ambulance service targets an average response time of 8 minutes; however, if a person is deprived of air for this duration, the likelihood of severe brain injury is significantly increased. According to data from the Office for National Statistics, approximately 351 individuals die annually in the UK as a direct result of choking. Experts suggest this figure likely underrepresents the true scale, as many choking-related deaths are attributed to underlying conditions such as Parkinson's disease, Alzheimer's disease, or Multiple Sclerosis, conditions that often impair swallowing and increase choking risk. Having a LifeVac on your premises will ensure that both staff and visitors will be protected in the case of a choking accident, be that on food, medications, dental appliances or everyday items. **Key Benefits of This Anti-Choking Device**

- **Multiple mask sizes included** – four different mask sizes are supplied within the LifeVac Wall-Mounted Kit to provide a secure seal for both paediatric and adult patients.
- **Practice mask provided** – to build confidence and familiarity, the kit includes a dedicated mask for training before an emergency occurs.
- **Regulated Medical Device** – CE marked, UKCA marked, MHRA regulated and MDR compliant.
- **One-way valve and suction system** – offers non-invasive airway clearance while reducing the risk of pushing obstructions deeper, protecting both the casualty and responders
- **No power or batteries required** – always ready for immediate use, even during power outages.
- **Free replacement offer** – in the event the LifeVac device is used, you can apply for a free replacement by submitting a form on their website.
- **Stored in an Evolution case** – this version of the LifeVac kit is housed in a durable Evolution case, ideal for busy workplaces with little storage spaces.

The Evolution Case The bold yellow Evolution case is specially designed to protect the contents of the LifeVac kit. Manufactured in the UK, it reduces unnecessary carbon emissions and is fully recyclable when replacement is needed. The case includes a wall-mountable bracket that can be securely fixed using screws (not supplied). This ensures the kit remains highly visible and easily accessible to everyone in the vicinity. Additionally, the bracket design makes it immediately apparent if the kit has been removed by displaying a 'Choking Rescue Kit Missing' message. Mounting the kit on the wall significantly reduces response time during emergencies by eliminating the need to search for supplies, enabling rapid access when every second counts. **The Kit Includes:**

- LifeVac Instruction Booklet – The instruction booklet details how to use LifeVac, and which coloured sticker identifies the size of the mask.
- LifeVac Unit.
- 1 Paediatric Mask.
- 1 Small Child/Adult Mask.

Product Specifications

- 1 Medium Adult (test) Mask.
- 1 Test Mask
- 1 Large Adult Mask.
- A3 BLS Poster (Basic Life Support)
- 1 Instruction for Use Leaflet
- 1 Window/Door Sticker – to easily identify where a LifeVac is stored and inform customers that a LifeVac is available on the premises.
- Feedback Form – The Feedback Form is to be filled out once the device is used to help LifeVac raise awareness.

How to use a LifeVac 1. Push and twist the mask into the base of your LifeVac unit. 2. Hold the mask firmly in place over their mouth and nose with your less dominant hand, making sure their head is tilted back. 3. Use your dominant hand on the LifeVac handle, depress and then pull upwards with a short, swift tug (while holding the mask firmly in place). 4. Roll the person onto their side, check their mouth to clear any debris, and repeat the process up to five times if the obstruction is not cleared. **Ideal for:**

- Restaurants and kitchens
- Care homes and assisted living facilities
- Healthcare clinics and rehabilitation centres
- Schools and childcare centres
- Workplaces with at-risk employees
- Community centres and public venues



INSTRUCTIONS FOR USE IN THE CASE OF CHOKING

If a casualty is a victim of choking

- 1** Encourage the victim to cough.
- 2** If the cough is not effective, call Emergency Services and give 5 Back Slaps.
- 3** If obstruction has not been cleared, follow by 5 Abdominal Thrusts.




Once unconscious, or if steps applied, use LIFEVAC

Place over mouth and nose creating a good seal with correct sized mask

PLACE



PUSH



REPEAT UNTIL BLOCKAGE IS REMOVED OR UP TO

If blockage is not removed commence CPR

UK CA CE

20196 Europe Limited
Unit 3 Torridge Business Park, Exeter Lane,
South Molton, Devon, EX36 4BP United Kingdom
Telephone: +44 (0)1393 31044
M06: 020 40 0000375

CE

010 Medical Devices, CE marked in accordance with
MDD 01/03/2002/019

UK CA

Best Care Consulting Group, Northbrook 115
211 610, Alderholt, Southampton
M06: (0)1483 2000963



LIFEVAC EUROPE LTD

Rev. 5.1 01 OCT 2024




Instructions For Use

1. Unconscious or if steps 1-3 are not effective, call emergency services and give 5 back slaps and 5 abdominal thrusts. If blockage is not removed, commence CPR.

2. Place LIFEVAC over the victim's mouth and nose, ensuring a good seal is created. Push down on the mask to create a seal.

3. Push down on the mask to create a seal.

4. Repeat until blockage is removed or up to 5 attempts. If blockage is not removed, commence CPR.

5. If blockage is not removed, commence CPR.

Customer Feedback Form

We value your feedback and we are always looking for ways to improve our products. Please take a few minutes to complete this form and return it to us. Your feedback will help us to improve our products and services. Thank you for your feedback.

1. Which were your contributions and would you like to see your (LIFEVAC) used?
2. How many patients did you use LIFEVAC on? (Please specify)
3. How long did you use LIFEVAC for? (Please specify)
4. How many times did you use LIFEVAC? (Please specify)

Name: _____ Telephone: _____

Address: _____ Email: _____

Rev 5.1 01 OCT 2024



LifeVac[®] Choking Rescue Contents

Items	Qty
Paediatric Mask	1
Small Child/Adult Mask	1
Medium Adult Mask	1
Large Adult Mask	1
LifeVac[®] Anti-Choking Device	1
Practice Mask (Medium Adult)	1
BLS Poster, A3	1
Window/Door Sticker	1
Instructions For Use Leaflet	1
LifeVac[®] Instruction Booklet	1



Child and adult masks included





